

# *PerfectPearl*<sup>™</sup> Disposable Prophylaxis Angle with Latex-Free Cup

## **PerfectPearl<sup>™</sup> Disposable Prophylaxis Angle**

The PerfectPearl disposable prophylaxis angle is a unique DPA with a feature set designed for maximum control and maximum comfort.

- Patented Swiss-designed prophylaxis cup propels paste toward tooth for cooling and effectiveness. Also has unique lip that flares easily for great subgingival access.
- Contra-angle design with bevel gears and straight shaft for super comfort and smoothness.
- Tactile grip pad for excellent control under slippery conditions.

## **Indications**

PerfectPearl<sup>™</sup> disposable prophylaxis angles are designed for single use only as part of a professionally administered prophylaxis treatment. This angle is designed to be compatible with hand pieces that use Doriot style attachments meeting ISO 3964.

## **Contraindications**

None known.

## **Warnings**

Air pressure may vary, it is recommended to use the least amount of air pressure to provide sufficient torque.

## **Precautions**

The PerfectPearl disposable prophylaxis angles are designed for single use only. Do not reuse.

## **Directions for use**

1. Remove the PerfectPearl disposable prophylaxis angle from the package by tearing open at the notches.
2. Insert the PerfectPearl disposable prophylaxis angle onto the hand piece. Make sure the notch at the base of the prophylaxis angle lines up with the pin on the hand piece.
3. Tighten the retention mechanism on the hand piece to hold the prophylaxis angle in place.
4. Operate at a low speed setting of 1,500 to 3,000 RPM.
5. Dispose of the PerfectPearl disposable prophylaxis angle after each patient, in accordance with federal, state, and local regulations.



Manufactured for  
Metrex<sup>®</sup> Research Corporation  
28210 Wick Road  
Romulus, MI 48174 USA  
800.841.1428  
Kerrtotalcare.com

70-9516